
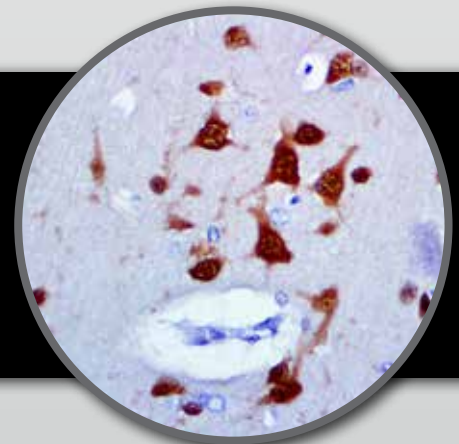
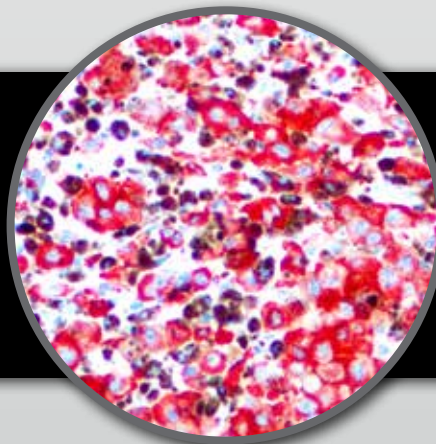
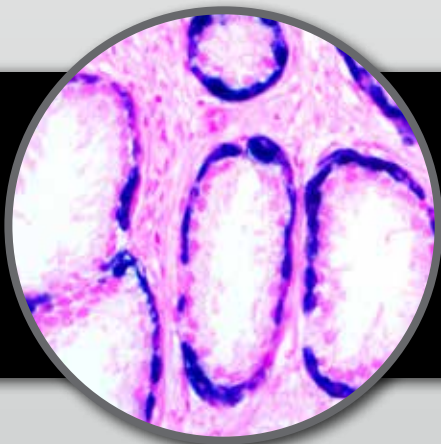


TintoRetriever PT Module

Epitope Retrieval System for use in Immunohistochemistry

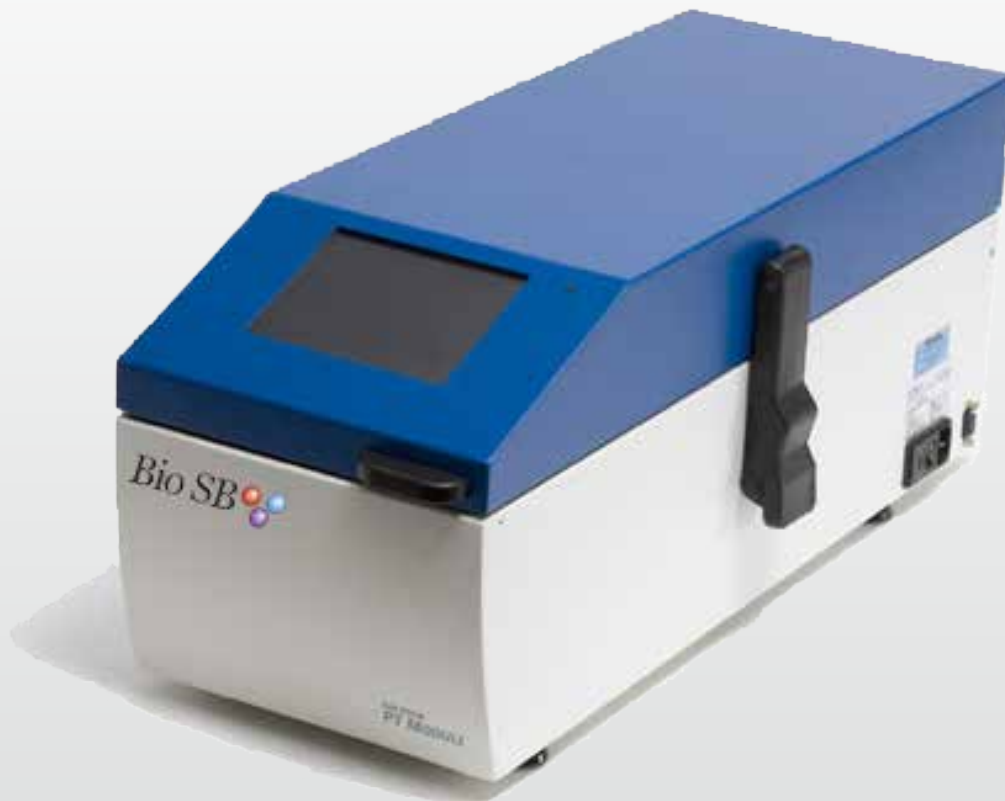


The Bio SB TintoRetriever PT Module

Most formalin-fixed paraffin-embedded tissues require an antigen retrieval step prior to any immunochemical staining. The antigen retrieval step breaks some of the methylene bridges that cross-link proteins in formalin-fixed tissue and allow antibodies to bind to unmasked epitopes.

Heat induced epitope retrieval and nucleic acid retrieval steps are most often performed using a conventional microwave, steamer, water bath or a pressure cooker. The Bio SB TintoRetriever PT is a precision-controlled heat source capable of maintaining a constant and reliable temperature while minimizing the potential for evaporation of the working solution under HIER procedure conditions. The TintoRetriever PT Module is recommended for use in both Immunohistochemistry (IHC), Immunocytochemistry (ICC) and In-Situ Hybridization (ISH) protocols.

TintoRetriever PT Module Overview


- Intuitive and programmable touch interface.
- Slides can be quickly transferred from TintoRetriever PT Module to the TintoStainer.
- Generates epitope/nucleic acid retrieval report.
- Ideal for automated work flow environments.

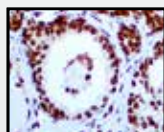

IHC
p40 RMAb
BSB 2072



FISH
ALK/EML4 TriCheck
Z-2117-200

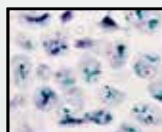

CISH
HER2/CEN 17
C-3022-40

- Rapid heat retrieval
- Intuitive touch screen interface
- Multiple temperature settings
- Use in IHC & ISH applications




Product	Catalog
TintoRetriever PT Module (110V)	BSB 7030
TintoRetriever PT Module (220V)	BSB 7033
Plastic Staining Dish	BSB 7009
Slide Holder (24 Slides)	BSB 7010
TintoRetriever Slide Thermometers	BSB 7005

TintoStainer Features

- Integrated bar-code scanning reduces user-error.
- Flexible protocol programming.
- Ideal for both research and clinical settings.
- Allows the use of antibody, reagent, and detection chemistry from any supplier.

Product	Catalog
TintoStainer 36 Slide (110v) with Report & Bar code Printers	BSB 7029
TintoStainer 36 Slide (240v) with Report & Bar code Printers	BSB 7032
TintoStainer 48 Slide (110v) with Report & Bar code Printers	BSB 7051
TintoStainer 48 Slide (240v) with Report & Bar code Printers	BSB 7074
TintoStainer 72 slide (110v) with Report & Bar code Printers	BSB 7052

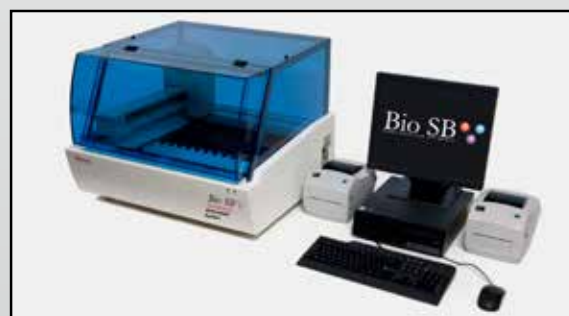

Ordering

United States

Phone: 1-800-561-1145

International

Phone: 1-805-692-2768

Global Distribution

www.biosb.com/distributors