

## TintoRetriever Pressure Cooker Staining Dish Support



IVD

**Bio SB**  
BIOSCIENCE FOR THE WORLD

www.biosb.com

### Summary and Explanation

The **TintoRetriever Pressure Cooker Staining Dish Support** allows for support and easy handling of the staining dishes and slide holder while using the TintoRetriever Pressure Cooker. The **TintoRetriever Pressure Cooker Staining Dish Support** holds 4 staining dishes and slide holders, allowing for easy handling of 96 total microscope slides to be used in histological, cytological, immunohistochemistry, Immunofluorescence, Mohs IHC and in situ hybridization procedures. The **TintoRetriever Pressure Cooker Staining Dish Support** provides added stability for the Staining Dish, and can be used to handle Slide Holders for Heat-Induced Epitope and Nucleic Acid Retrieval using the Bio SB TintoRetriever Pressure Cooker (BSB 7008). The TintoRetriever Pressure Cooker Staining Dish Support is autoclavable.

### Presentation

The **TintoRetriever Pressure Cooker Staining Dish Support** is made of grade 316 stainless steel.

Product Name	Catalog No.
TintoRetriever Pressure Cooker Staining Dish Support	BSB 7086

### Storage

Store at 20-25°C

### Stability

Stable at room temperature

### Precautions

1. Use caution when items are hot to prevent scalding and burns.
2. Do not place Staining Dish directly on heat source, as it may melt.
3. For professional users.
4. Avoid any contamination.

### Protocol

1. Cut and mount formalin-fixed paraffin-embedded tissues, frozen tissues or cells on microscope slides.
2. Mount up to 24 slides in the Slide Holder.
3. Immerse the Slide Holder in the Staining Dish containing the treating solution and incubate or treat as needed.
4. Place the Staining Dish on **TintoRetriever Pressure Cooker Staining Dish Support** for heat treatment in the Bio SB TintoRetriever Pressure Cooker.

### Symbol Key / Légende des symboles/Erläuterung der Symbole

	In Vitro Diagnostic Medical Device Dispositif médical de diagnostic in vitro In-Vitro-Diagnostikum		Storage Temperature Limites de température Zulässiger Temperaturbereich		Manufacturer Fabricant Hersteller		Catalog Number Référence du catalogue Bestellnummer
	Read Instructions for Use Consulter les instructions d'utilisation Gebrauchsanweisung beachten		Expiration Date Utiliser jusque Verwendbar bis		Lot Number Code du lot Chargenbezeichnung		

**Bio SB**  
BIOSCIENCE FOR THE WORLD



69 Santa Felicia Dr., Santa Barbara, CA 93117, USA  
Tel. (805) 692-2768 | Tel. (800) 561-1145 | Fax. (805) 692-2769  
E-mail: sales@biosb.com | Website: www.biosb.com