

Clear-it™ Stringency Buffer

REF WB-0009-500

Σ 7 (500 ml)

For use in *in situ* hybridization procedures

CE

IVD

In vitro diagnostic medical device

according to EU directive 98/79/EC



1. Scope of Application

This product is designed for *in vitro* diagnostic use (according to EU directive 98/79/EC). Interpretation of results must be made within the context of the patient's clinical history with respect to further clinical and pathologic data of the patient by a qualified pathologist!

The Clear-it™ Stringency Buffer (**WB9**) is designed to be used for washing steps in *in situ* hybridization (ISH) protocols of ZytoVision.

2. Safety Precautions and Disposal

- ✓ Read the operating instructions prior to use!
- ✓ Do not use the reagents after the expiry date has been reached!
- ✓ Avoid any cross-contamination and micro-bacterial contamination of the reagents!
- ✓ Avoid any direct contact with the reagents. Take appropriate protective measures (use disposable gloves, protective glasses, and lab garments)!
- ✓ If reagents come into contact with skin, rinse skin immediately with copious quantities of water!
- ✓ Never pipet solutions with your mouth!
- ✓ The disposal of reagents must be carried out in accordance with local regulations!
- ✓ A material safety data sheet is available on request for the professional user!

3. Package Content

Code	Component	Quantity	Container
WB9	<u>Clear-it™ Stringency Buffer</u>	500 ml	Screw-cap bottle (large)
	Instruction manual	1	

Component (**WB9**) is sufficient for 7 staining jars of 70 ml each.

4. Storage and Shelf Life

The component must be stored at 2...8°C. If these storage conditions are followed, the component will function, without loss of performance, at least until the expiry date printed on the label.

5. Instructions

Clear-it™ Stringency Buffer (WB9) is a ready-to-use solution for washing steps in ISH applications, following the ISH protocols of ZytoVision.

Following the ISH protocols of ZytoVision the Clear-it™ Stringency Buffer (WB9) is replacing the Wash Buffer SSC (WB1) for an improved stringency wash.

Our experts are available to answer your questions.


As of: January 2, 2013 (5.0)

Trademarks:

ZytoVision® is a trademark of ZytoVision GmbH.



ZytoVision GmbH · Fischkai 1
D - 27572 Bremerhaven · Germany
Phone: +49 (0) 471/4832 - 300
Fax: +49 (0) 471/4832 - 509
www.zytovision.com
info@zytovision.com

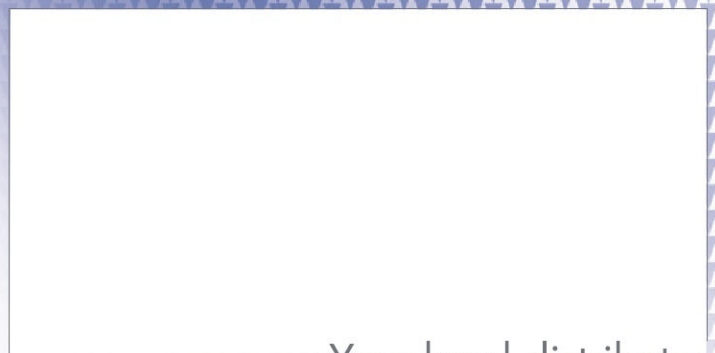

Your local distributor