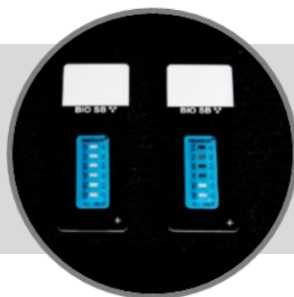


# TintoRetriever Slide Thermometers, 5 slides


**IVD**
**Bio SB**  
 BIOSCIENCE FOR THE WORLD

[www.biosb.com](http://www.biosb.com)

## Summary and Explanation

The TintoRetriever Slide Thermometers are intended as a quality control tool to evaluate the temperature of Retrieval solutions, such as the Bio SB ImmunoDNA Retriever Citrate (BSB 0020 – BSB 0023) and EDTA (BSB 0030 – BSB 0033), used in cell or tissue thermal permeabilization procedures for Immunocytochemistry (ICC), Immunohistochemistry (IHC), Chromogenic in Situ Hybridization (CISH), Fluorogenic in situ Hybridization (FISH) or in situ PCR.

The TintoRetriever Slide Thermometers can be used with pressure cookers, microwaves, steamers, water baths and other sources of heat. They have the ability to record the highest temperature in both Centigrade (99, 104, 110, 116, 121 and 127 °C) and Fahrenheit (210, 219, 230, 241, 250 and 261 °F) degrees.

## Presentation

The TintoRetriever Slide Thermometers are supplied in slide mailers of 5 slides.

<i>Catalog No.</i>	<i>Number Provided</i>
BSB 7005	1

## Size

25mm x 75mm x 1mm

## Storage

Store at 20-25°C

## Stability

Stable at room temperature





## Precautions

For professional users. Avoid any contamination. Proper handling procedures should be used.

## Protocol

1. Immerse one TintoRetriever Slide Thermometer in Retrieval solution along with other slides containing cells or tissues that are to be tested.
2. Subject to thermal permeabilization using a pressure cooker, microwave, steamer, water bath, or another source of heat prior to ICC, IHC, CISH, FISH or in situ PCR procedures.
3. After the thermal permeabilization procedure, take the TintoRetriever Slide Thermometer out of the Retrieval solution, let it dry and read it. The temperature is the highest temperature reading turning black.

## Symbol Key / Légende des symboles/Erläuterung der Symbole

<b>IVD</b>	In Vitro Diagnostic Medical Device Dispositif médical de diagnostic in vitro In-Vitro-Diagnostikum	 Storage Temperature Limites de température Zulässiger Temperaturbereich	 Manufacturer Fabricant Hersteller	<b>REF</b>	Catalog Number Référence du catalogue Bestellnummer
		 Read Instructions for Use Consulter les instructions d'utilisation Gebrauchsanweisung beachten	 Expiration Date Utiliser jusque Verwendbar bis	<b>LOT</b>	Lot Number Code du lot Chargenbezeichnung

**Bio SB**  
 BIOSCIENCE FOR THE WORLD


69 Santa Felicia Dr., Santa Barbara, CA 93117, USA  
 Tel. (805) 692-2768 | Tel. (800) 561-1145 | Fax. (805) 692-2769  
 E-mail: [sales@biosb.com](mailto:sales@biosb.com) | Website: [www.biosb.com](http://www.biosb.com)